

GENERAL ENVIRONMENTAL HYGIENE

Actichlor™

Chlorine Disinfectant Range

Comprehensive range of chlorine disinfectant products for all aspects of surface and environmental disinfection.

Actichlor also provides:

- Full range of formats and concentrations
- Broad spectrum of effects
- Simple to use



ENVIRONMENTAL HYGIENE

Actichlor™

Chlorine Disinfectant Range

Why Use Actichlor™?

Comprehensive range of chlorine disinfectant products for all aspects of surface and environmental disinfection:

- Full spectrum of efficacy
- Wide range of tablet sizes available

MICROBIOLOGICAL PROPERTIES

Disinfectant solutions made with Actichlor are effective against all bacteria, spores, yeasts, moulds and viruses including Hepatitis B and HIV.

Tested to EN 1276, EN 1650, EN 13704 and EN 13713.

EXCELLENT CLEANING AND DISINFECTION PERFORMANCE IN ONE EASY STEP

The Actichlor Plus tablet combines a chlorine compatible detergent with NaDCC in a single tablet format.

PRODUCT SPECIFICATIONS

Actichlor Disinfectant Tablets - Each Actichlor tablet contains Sodium Dichloroisocyanurate (NaDCC, troclosene sodium) in an inert effervescent base.

Actichlor Granules - Absorbent NaDCC granules for the disinfection of body fluid spills.

Actichlor Sodium Hypochlorite Solution - Stabilised Sodium Hypochlorite Solution (1%).

Actichlor Plus Tablets - Effervescent NaDCC tablets with detergent.

Actichlor Spill Kit - 10 clean up packs, with dilution bottle and disinfection tablets for blood spills.

PROVEN. INNOVATIVE. EFFICIENT.

APPLICATION AREA

For surface disinfection of blood spills, general clinical environment, instruments, sinks and drains, babies' bottles, catering equipment.

Use Actichlor Spill Kit or Actichlor Granules on blood spills.

INSTRUCTIONS FOR USE

To achieve a solution of 1000ppm, dissolve 1 tablet in 1 litre of water; or for a solution of 5000ppm, 5 tablets in 1 litre of water.

INGREDIENTS

Each Actichlor tablet contains Sodium Dichloroisocyanurate (NaDCC, troclosene sodium) in an inert effervescent base.

Actichlor Spill Kit - 10 clean up packs, with dilution bottle and disinfection tablets for blood spills.

PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Actichlor Plus 1.7g Disinfectant Tablets	1.7g x 150	3029060
Actichlor Foil Disinfectant Tablets	0.5g x 80	3050740
Actichlor 0.5g Disinfectant Tablets	0.5g x 250	3049690
Actichlor 0.5g Disinfectant Tablets	0.5g x 600	3049680
Actichlor 1.7g Disinfectant Tablets	1.7g x 30	3049170
Actichlor 1.7g Disinfectant Tablets	1.7g x 200	3037640
Actichlor 2.5g Disinfectant Tablets	2.5g x 100	3049670
Actichlor 5g Disinfectant Tablets	5g x 50	3037640
Actichlor 500g Granules	500g	3037680
Actichlor Spill Kit	1	3030520
Actichlor Dilution Bottle	1	408432



Actichlor Plus 1.7g Disinfection Tablets



Actichlor 0.5g Disinfection Tablets



Actichlor 1.7g Disinfection Tablets



Actichlor Granules



Actichlor Spill Kit

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE.

HEALTHCARE DIVISION
Lotherton Way, Garforth
Leeds LS25 2JY, UK
+44 (0)113 232 0066
www.ecolab.co.uk

La Vallée House, Upper Dargle Road
Bray, County Wicklow, Ireland
+353 (0) 276 3500